Bitte Ihrem Kunden übergeben.
Please submit your customers.
A remettre à votre client.
Aub aan uw klant overhandigen.
Da consegnare al cliente finale.

Artweger[®]

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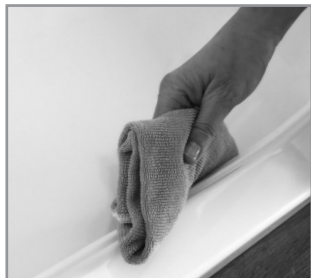
DEAR CUSTOMER ■ ■

**CONGRATULATIONS - YOU HAVE CHOSEN
A FIRST CLASS QUALITY PRODUCT
FROM ARTWEGER.**

ARTLIFT FROM ARTWEGER!

Artweger ARTLIFT is a shower, bathtub and a seat lift all in one and enables comfortable, barrier-free showering and bathing.

The integrated seat lift is extremely space saving and functionally designed so that you can still feel relaxed, unrestricted and undisturbed in your bathtub. After bathing you can simply rise up at the push of a button!

We wish you much joy with your new Artweger ARTLIFT!

Keep this brochure in a safe place in case you need to refer to it again.

FUNCTION ELEMENTS

Handle Handle and piece Bathtub taps

Shutter unit

Operating unit

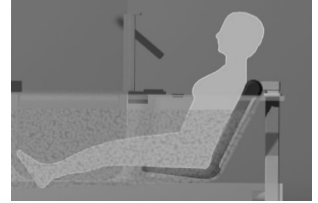
Shower door
(opens inwards)


Seat lift

BATHING WITH ARTLIFT

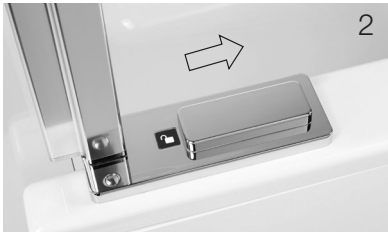- 1.) Move the seat lift to the upper position. To do this, press the arrow "seat up" (fig.1) continually until the seat stops at its upper position. The seat symbol on the control unit is lit in blue as long as a button is pushed.

- 2.) Fold the seat of the seat lift down.
- 3.) Set the locking unit to "shower position" (fig. 2) – The door is not locked, the tub drain is open. 

- 4.) By pulling the end of the handle upwards (fig. 3) you can open the door inwards. Get into the tub.

- 5.) When the shower door is closed it automatically fits into place and remains fixed in this position.
- 6.) Set the locking unit to "bath position" (fig. 4). Now the door is locked and it cannot be opened. At the same time the drain is shut off and you can fill the bathtub. 

- 7.) Sit down on the seat lift. The press the arrow "seat down" (fig.1) and the seat begins to move downwards. Keep the key pressed while the seat is lowering. The seat lift stops automatically at its lowest position; then you can release the key.

ATTENTION: When the seat lift is in motion stretch your legs forward and place your hands on the sides of the tub.

- 8.) After bathing set the locking unit back to "shower position" (fig. 2). The drain is open again, the water can run out and the door is unlocked.

- 9.) Press the arrow "seat up" (fig. 1) continually. The seat starts to move and stops automatically at its uppermost position. While the seat is moving the seat symbol is lit again.

ATTENTION: Wait until all the water has run out before opening the door!

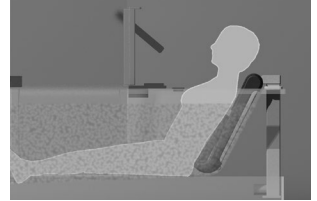
- 10.) When you stand up please be careful of good footing. (Danger of slipping).
- 11.) Now open the shower door by lifting the end of the handle (fig. 3) and step out of the tub.
- 12.) The seat can be folded back up. If desired the seat can now be returned to its lower position.

Maximum load for seat: 120 kg

BATHING WITH THE ARTLIFT SEAT LIFT AS BACKREST

- 1.) Set the locking unit to "shower position" (fig. 1) – The door is not locked, the tub drain is open. 
- 2.) By pulling the end of the handle upwards (fig. 2) you can open the door inwards. Get into the tub.
- 3.) When the shower door is closed it automatically fits into place and remains fixed in this position.
- 4.) Set the locking unit to "bath position" (fig. 3). Now the door is locked and it cannot be opened. At the same time the drain is shut off and you can fill the bathtub. Fold the seat up. You can now use the seat lift as a backrest. 
- 5.) After bathing set the locking unit back to "shower position" (fig. 1). The drain is open again, the water can run out and the door is unlocked. 

ATTENTION: Wait until all the water has run out before opening the door!

SHOWERING WITH THE ARTLIFT

- 1.) Set the locking unit to "shower position" (fig. 1) – The door is not locked,  the tub drain is open.
- 2.) By pulling the end of the handle upwards (fig. 2) you can open the door inwards. Get into the tub.
- 3.) When the shower door is closed it automatically fits into place and remains fixed in this position.
- 4.) After showering lift the end of the handle upwards and open the door (fig. 2).

Tip: Clean the shower before leaving the tub.

OPERATION OF THE DIVIDED DOOR

- 1.) By lifting up the grip cover you uncouple the shower section from the tub door. (Fig. 1)
- 2.) Move the shower section inwards or outwards. (Fig. 2)
- 3.) When closing, pull the grip cover up again and lower it over the tub handle. (Fig. 1)
Now the shower section and the tub door are combined again.

Operation of the bathtub door can be found on the next page.

OPERATION OF THE BATHTUB DOOR

- 1.) Move the lock unit to "Duschposition" ("Shower Position"). The door is not locked, the tub drain is open.
Lift the door handle up gently (Fig. 1).

- 2.) Tub door opens inwards (Fig. 2).

- 3.) To close the tub door let it snap shut. Move the lock unit to "Badeposition" ("Bath Position"). The door is locked, the tub drain is closed (Fig. 3).

ATTENTION: After bathing, wait until all the water has drained out before opening the door! Do not leave children or persons with learning disabilities unsupervised!

GUARANTEED SAFETY

At Artweger not only are all the materials used tested but also all production processes monitored and the function of the products inspected. Regular testing for water tightness and resistance to blows as well as load durability tests guarantee stable high quality. **All Artweger products are tested in accordance with current standards for stability, water tightness and durability..**

AVAILABILITY GUARANTEE 10 YEARS

For spare parts.

TÜV TESTED

Valid for Airpool.

GENERAL GUIDELINES

This device may be used by children of 8 years and above, as well as by persons with reduced physical, sensory or mental capabilities or with a lack of experience and knowledge, as long as they are being supervised, or have been instructed on how to use the device and understand the potential dangers of it. Children are not allowed to play with the device. Cleaning and user maintenance must not be performed by children without supervision. If the device malfunctions, the appropriate customer service should be called and only this person or a qualified electrician should perform repair or maintenance work on or under the bathtub.

The owner of the device is requested to hand on to any further owner the operating instructions and the details of the specialist who installed the device.

Possible danger in the case of lack of concentration or danger for another person: Being squeezed between the roller and the tub, back-rest and tub, head plate and tub side or seat and tub in the lowest position. Possibility of hair becoming entangled between roller and tub.

No service intervals have been foreseen.

Type plate on the back of the junction box.

Technical data:

ARTLIFT: requires a 230V power connection. The control unit and the motor function on a low voltage of 24V.

230V 50 HZ, consumption: 120 W

ARTLIFT with colored lighting

230V 50 HZ, consumption: 130 W

Maximum load for seat: 120 kg

Intended use:

Suitable for use in Artweger ARTLIFT tubs. For the purpose of raising and lowering one person in private households.

Applicable documents:

Pre-installation sheet (11768); Installation instructions ARTLIFT tub (11771); Installation instructions ARTLIFT door (11773); Installation instructions seat lift (11769).

CARE

PLEASE FOLLOW THE CARE INSTRUCTIONS BELOW SO THAT YOUR ARTWEGER ARTLIFT RETAINS ITS GREAT LOOK FOR MANY YEARS:

The seat lift is not suitable as a shelf for shower gels or cosmetics. Rinse off all surfaces with clear water after use. If used fold seat down to rinse. Remove any remaining water with a scraper or a soft cloth. Do not clean in a dry state!

For hygienic and easy cleaning of your Artweger ARTLIFT we recommend Artweger BLITZBLANK, a bio-degradable cleaner which is available from your bathroom specialist.

Regular cleaning with Artweger BLITZBLANK prevents the buildup of persistent residues and ensures a clean and hygienic surface for many years.

Important! Under no circumstances should you use lime descaler, abrasive or caustic cleaners, microfibre cloths or steam cleaners! For chrome parts please use only cleaners expressly declared as suitable for cleaning chrome parts! The seal on the shower door and the tub are designed as maintenance joint and are therefore not covered by the guarantee.

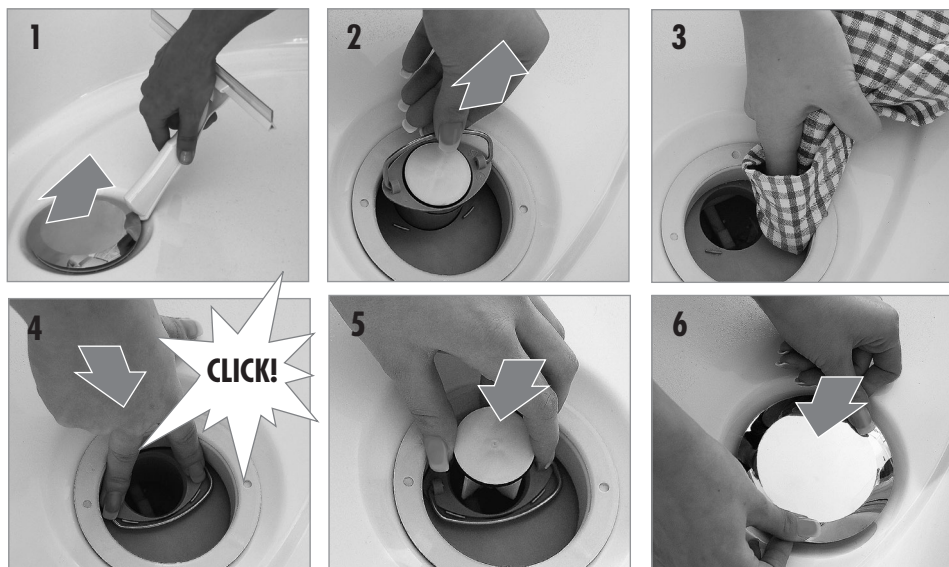
The guarantee is only valid when the cleaning guidelines are followed.

Do not dispose of the device in household garbage after its useful life. The device has to be disposed of according to environment protection regulations. Please enquire at your local waste disposal authority.


Cleaning the drain:

Cleaning the pan gasket:

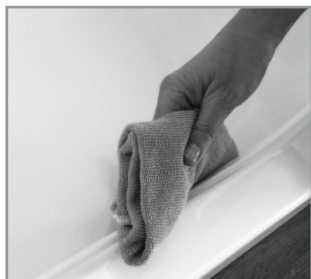For the permanent leak tightness of the door system, we recommend wiping the pan gasket regularly with a damp cloth to remove any residues (dust, etc.).

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Declaration of Conformity EN

Declaration of Conformity EN No.: 07.0 1

In accordance with EG Machine Guideline 2006/42/EG from 17 May 2006

E declare herewith that the article described below complies with the basic health and safety requirements of the EG Guideline 2006/42/EG in its design, construction as well as in the form brought onto the market by us. This declaration is no longer valid should any changes be made to the article without express permission from us.

Manufacturer: Artweger GmbH. & Co. KG, Sulzbacherstr. 60, 4820 Bad Ischl, Austria

Identification of the article: Artlift

Intended use: Intended for use in Artweger ARTLIFT bathtubs.

To raise and lower a maximum of one person in private households.

Product group: Household appliances

Model: Artlift, Artlift with LED colored lighting

Year of manufacture: 2013

The declaration of conformity is also made for further guidelines and standards applicable to the product:

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