Bitte Ihrem Kunden übergeben.
Please submit your customers.
A remettre à votre client.
Aub aan uw klant overhandigen.
Da consegnare al cliente finale.

Artweger[®]

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DEAR CUSTOMER ■ ■

**FOR THE OPERATION AND CARE OF YOUR
ARTLIFT SHOWER BATHTUB AND YOUR
SEAT LIFT PLEASE SEE THE RESPECTIVE
OPERATION INSTRUCTIONS.**

ARTLIFT FROM ARTWEGER!

You have made the choice of an ARTLIFT shower-bathtub already fitted with the original pre-installation set. This choice allows you to retro-fit an ARTLIFT seat lift at any time.

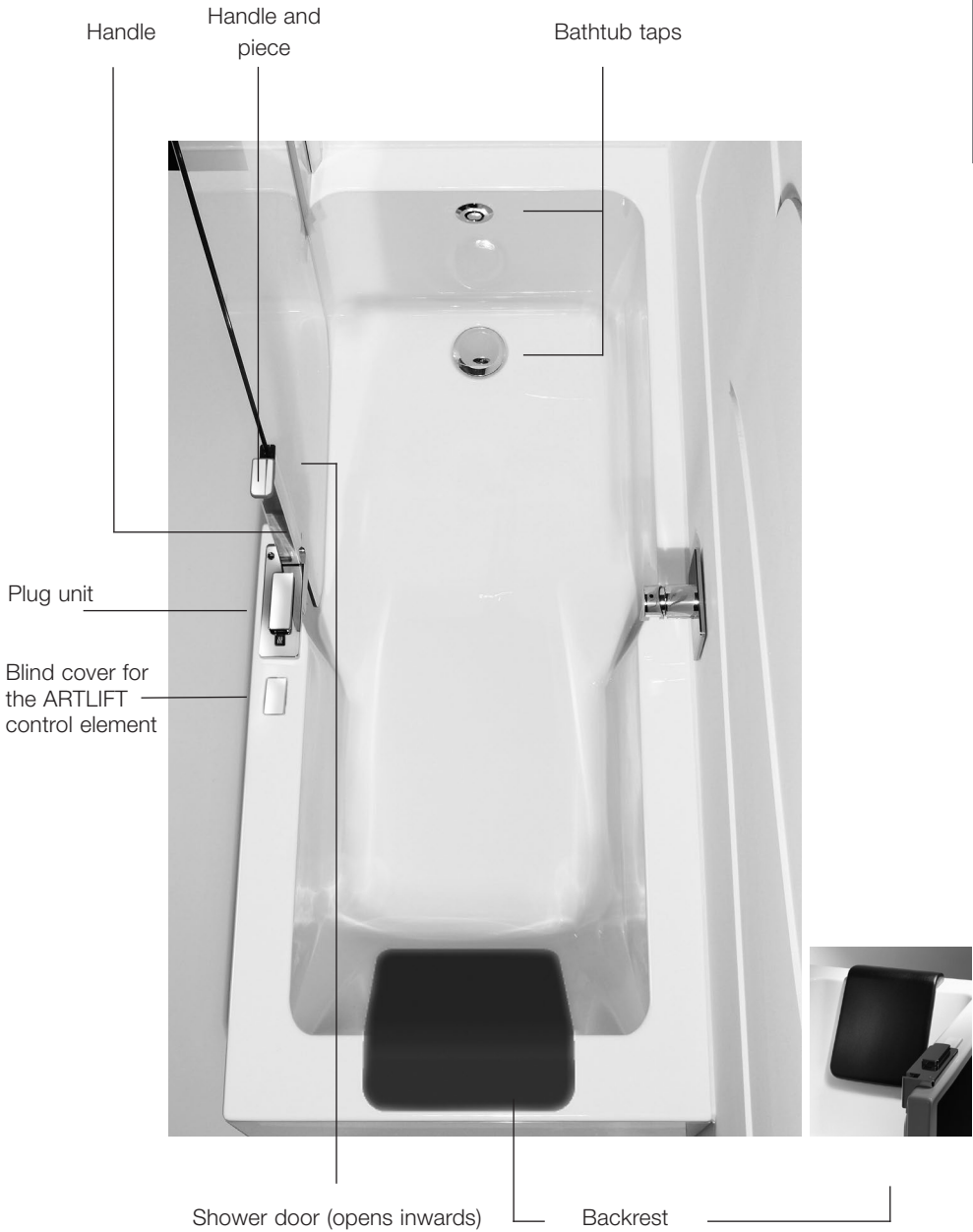
In the case of the pre-installation set the hollow for the hoist mechanism is concealed by a comfortable chrome cover.

**WE WISH YOU MUCH JOY WITH YOUR NEW
ARTWEGER ARTLIFT!**

Keep this brochure in a safe place in case you need to refer to it again.

SYSTEM ILLUSTRATION

EN

SHOWERING

1.) Move the closure unit to "shower position": The door is not locked, and the tub drain is open (fig. 1). 

2.) Open the shower door to the inside by pulling up the handle end (fig. 2).

3.) The door engages automatically when closing and stays fixed in this position.

4.) After the shower, pull the handle end up briefly and in this way open the door (fig. 2).

Tip: Clean the shower before you get out of the Artweger ARTLIFT

BATHING

ZEN

1.) Move the closure unit to "shower position": The door is not locked, and the tub drain is open (fig. 1). 

2.) Open the shower door to the inside by pulling up the handle end (fig. 2).

3.) The shower door engages automatically when closing and stays fixed in this position.

4.) Move the closure unit to "bath position". Now the door is locked, and opening is no longer possible. At the same time the drains closed, and you can fill the bathtub (fig. 3) 

5.) After taking a bath set the closure unit to "shower position" again (fig. 1). Now the drain is opened again, the water runs out and the door is unlocked. 

ATTENTION: Wait with opening the door until the water has run out completely! Do not leave children and mentally impaired persons without supervision!

OPERATION OF THE DIVIDED DOOR

- 1.) By lifting up the grip cover you uncouple the shower section from the tub door. (Fig. 1)
- 2.) Move the shower section inwards or outwards. (Fig. 2)
- 3.) When closing, pull the grip cover up again and lower it over the tub handle. (Fig. 1)
Now the shower section and the tub door are combined again.

Operation of the bathtub door can be found on the next page.

OPERATION OF THE BATHTUB DOOR

EN

- 1.) Move the lock unit to "Duschposition" ("Shower Position"). The door is not locked, the tub drain is open.
Lift the door handle up gently (Fig. 1).

- 2.) Tub door opens inwards (Fig. 2).

- 3.) To close the tub door let it snap shut. Move the lock unit to "Badeposition" ("Bath Position"). The door is locked, the tub drain is closed (Fig. 3).

ATTENTION: After bathing, wait until all the water has drained out before opening the door! Do not leave children or persons with learning disabilities unsupervised!

GUARANTEED SAFETY

At Artweger not only the materials used are thoroughly checked but also the production processes and the functionality of the products. Regular leak tightness and shock testing as well as long-term load testing guarantee steadily high quality. **All Artweger products are tested in accordance with the standards applicable to stability, water-tightness and durability.**

REPLACEMENT GUARANTEE OF 10 YEARS

On spare parts.

RETRO-FITTING GUARANTEE OF 10 YEARS

For the original ARTLIFT seat lift.

GENERAL GUIDELINES

This device may be used by children of 8 years and above, as well as by persons with reduced physical, sensory or mental capabilities or with a lack of experience and knowledge, as long as they are being supervised, or have been instructed on how to use the device and understand the potential dangers of it. Children are not allowed to play with the device. Cleaning and user maintenance must not be performed by children without supervision.

If the device malfunctions, the appropriate customer service should be called and only this person or a qualified electrician should perform repair or maintenance work on or under the bathtub.

The owner of the device is requested to hand on to any further owner the operating instructions and the details of the specialist who installed the device.

No service intervals have been foreseen.

CARE

SO THAT YOUR ARTWEGER ARTLIFT RETAINS ITS ATTRACTIVE APPEARANCE FOR MANY YEARS, PLEASE NOTE THE FOLLOWING INSTRUCTIONS FOR ITS CARE:

After using, rinse all surfaces with clear water. The backrest (optional) is fixed in place with suction cups and can be removed for cleaning. Wipe away any droplets of water with the scraper or a soft cloth. Replace the backrest when everything is dry.

Do not clean the ARTLIFT in a dry condition! For simple and hygienic cleaning of your Artweger ARTLIFT we recommend the use of Artweger BLITZBLANK, a bio-degradable cleaner which you can get from your bathroom specialist.

Regular cleaning with Artweger BLITZBLANK prevents the build-up of hard to remove deposits and ensures clean and hygienic surfaces for many years to come.

Important! Under no circumstances should you use lime descaler, abrasive or caustic cleaners, microfibre cloths or steam cleaners! On chrome parts, please only use cleaning agents especially designated for use on chrome!

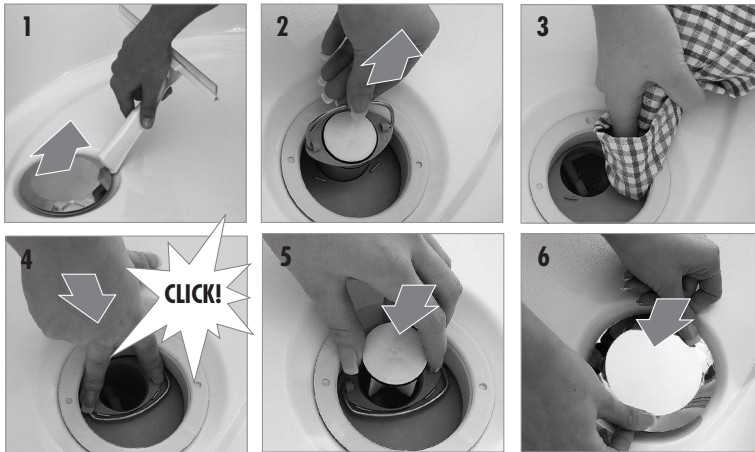
The seal between shower screen and bathtub is a seal which is to be regularly maintained and for this reason it does not fall under the guarantee.

The guarantee is only valid if the instructions for care have been complied with.

Do not dispose of the device with household waste at the end of its life. It has to be disposed of through the proper channels. Please enquire at your local disposal authority.

Cleaning the drain:

Cleaning the pan gasket:

For the permanent leak tightness of the door system, we recommend wiping the pan gasket regularly with a damp cloth to remove any residues (dust, etc.).

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Artweger GmbH. & Co. KG